



Cardio Calcium Scoring Insert

A calibration standard for cardiac CT.

Investigate the influence of scan parameters on the detectability of calcifications. Achieve reproducible scoring results.


The cylindrical cardiac calcification insert, QRM-CCI, contains nine cylindrical calcifications in variing size and hydroxyapatite (HA) density (see Table next side), and two larger calibration inserts. One of them is made of water equivalent material, the other one contains hydroxyapatite in addition. They are embedded in a tissueequivalent solid of typically 35 HU (+/- 5 HU) at 120 kV.

In order to obtain absolute values for the calcium mass, a calibration measurement with the central calibration insert of known hydroxyapatite (HA) density of 200 mg HA/cm³ can be carried out. The water equivalent insert serves as a further calibration standard. In practice the CT number of water should always be checked under the given modalities. If its measured HU value deviates from zero, the known deviation can be taken into account in the applied scoring method.

Specifications

Diameter of cardiac insert	100 mm
Hight of cardiac insert	100 mm
Weight of cardiac insert	880 g
Base material	resin



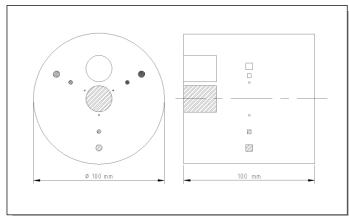

The Cylindrical Cardiac Calcification Insert, QRM-CCI with its three groups of small, radially arranged calcifications (see Table for details), and the two larger inserts for calibration purposes.

H A density	Length [mm]	Diameter [m m]	Volume [mm ³]	Area [m m ²]	H A m ass
$[m g/c m^3]$					[mg]
2 0 0	5.0	5.0	98.2	19.6	19.6
2 0 0	3.0	3.0	21.2	7.1	4.2
2 0 0	1.0	1.0	0.8	0.8	0.2
4 0 0	5.0	5.0	98.2	19.6	39.3
4 0 0	3.0	3.0	21.2	7.1	8.5
4 0 0	1.0	1.0	0.8	0.8	0.3
8 0 0	5.0	5.0	98.2	19.6	78.5
8 0 0	3.0	3.0	21.2	7.1	17.0
8 0 0	1.0	1.0	0.8	0.8	0.6

Properties of the nine cylindrical calcifications embedded in the Cardiac Calcification Insert QRM-CCI.

- **References** [1] McCollough C., Primak A., et al. Dose Performance of a 64-Channel Dual-Source CT Scanner. 2007, Radiology, Vol. 243, p.775-784.
 - [2] McCollough C., Ulzheimer S., et al. Coronary Atrery Calcium: A Multiinstitutunal, Multimanufacturer international Standard for Quantification at Cardiac CT. 2007, Radiology, Vol. 243, p. 527-538.